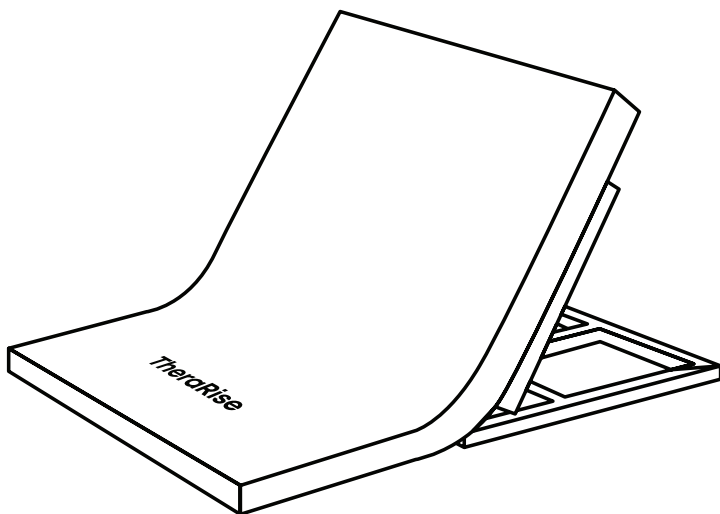
Respaldo reclinable automático – TheraRise^{MR}

MANUAL DE INSTRUCCIONES

IMPORTANTE: LEA DETENIDAMENTE Y CONSERVE.

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[EN] Thank you for choosing **TheraRise: Electric Adjustable Bed Backrest**. This equipment has been designed to transform your current bed into a dynamic, safe, and comfortable resting space. Specially developed to provide autonomy to seniors or those who need to spend extended periods in bed, TheraRise makes sitting up and shifting positions a natural, effortless process, reducing physical strain for both the user and caregiver, and promoting daily well-being with complete dignity.

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[EN] TheraRise is an Electric Adjustable Bed Backrest and motorized body support system placed directly on top of your current bed mattress. It features an integrated mattress with a non-removable protective cover and a heavy-duty articulating mechanism that smoothly elevates and lowers the back section using an easy-to-operate remote control. Its purpose is to provide customized ergonomic support that improves postural comfort, and allow daily activities such as reading, eating, or socializing to be done comfortably and safely.

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[EN]

- User Weight Restriction: Designed for users with a maximum body weight of up to 150 kg (330 lbs). Due to anatomical weight distribution on the bed, the torso exerts an operating force on the mechanism of up to 70 kg. Do not sit on or concentrate full body weight on the top section of the backrest when it is fully elevated.
- Avoid placing fingers or hands in or near moving parts (pinch points) during articulation.
- Plug directly into a surge protector (not included) or a suitable, grounded wall outlet.
- Keep power cords away from heated surfaces and any sources of moisture.
- Always unplug the backrest from the electrical outlet before cleaning or servicing.
- Do not drop or insert objects into any openings, and never operate the base when there is an obstruction between the moving parts of the adjustable base and the metal frame.
- Discontinue use of the bed base if it has a damaged cord or plug, or if it has been exposed to liquids.

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[EN] Carefully remove all packaging materials. Inspect the metal frame and the integrated mattress cover for any visible damage that may have occurred during shipping. Ensure there is sufficient clear space on your bed to unfold the base without obstruction.

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[EN]

1. No tools required: This product does not require tools for assembly.
2. Carefully unfold the base onto the existing bed, paying close attention to articulation points to prevent pinch injuries.
3. Securely plug in all power cord connections as outlined in the motor circuit diagrams.
4. Connect the wired remote control to the designated port located on the side of the motor.
5. Plug the power supply unit into a protected electrical outlet.

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[EN]

- Elevation: Press and hold the up-arrow button on the wired remote to smoothly raise the backrest to your desired angle (maximum of 58°).
- Recline: Press and hold the down-arrow button to return the backrest toward the flat resting position (minimum of 4.5°).
- Releasing either button will immediately stop the movement at the current position, ensuring a customized and safe adjustment.

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[EN] During typical operation, the wheels that allow the bed to articulate will make contact with the steel platform supports of the base, creating a contact noise. This noise may be heard when shifting weight, sitting up, or lying back down on the base, and noise levels can potentially change depending on the incline. This is normal behavior.

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[EN]

- Mandatory Disconnection: Always unplug the backrest system from the electrical outlet before cleaning.
- General surface cleaning: For general dirt, clean the surface with a soft cloth lightly dampened with water. For more ingrained dirt, a cloth with alcohol may be used. Dry the surface immediately with a dry cloth before reconnecting the device.

- Avoid Moisture: Under no circumstances should water or moisture come into contact with the motor, internal electronic components, remote control, or wiring. Do not use excessively wet cloths or steam cleaners.
- Do not use abrasive cleaning agents, solvents, or chlorine.

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[EN]

- If the adjustable bed backrest will not operate: Unplug the power cord from the outlet for 5 minutes to reset the internal electronic components. Afterward, plug it into a different electrical outlet, or verify the current outlet functions properly by testing it with another working appliance.
- If the mechanism elevates but will not return to the flat position (4.5°): Inspect the undercarriage and structural frame for obstructions. Remove any objects, bedding, or items away from the moving mechanisms to allow the base to return to its flat position.

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[EN] At TheraMart, we create tools that support life's cycles with respect and professional quality. Because when rest is secure, life is better.

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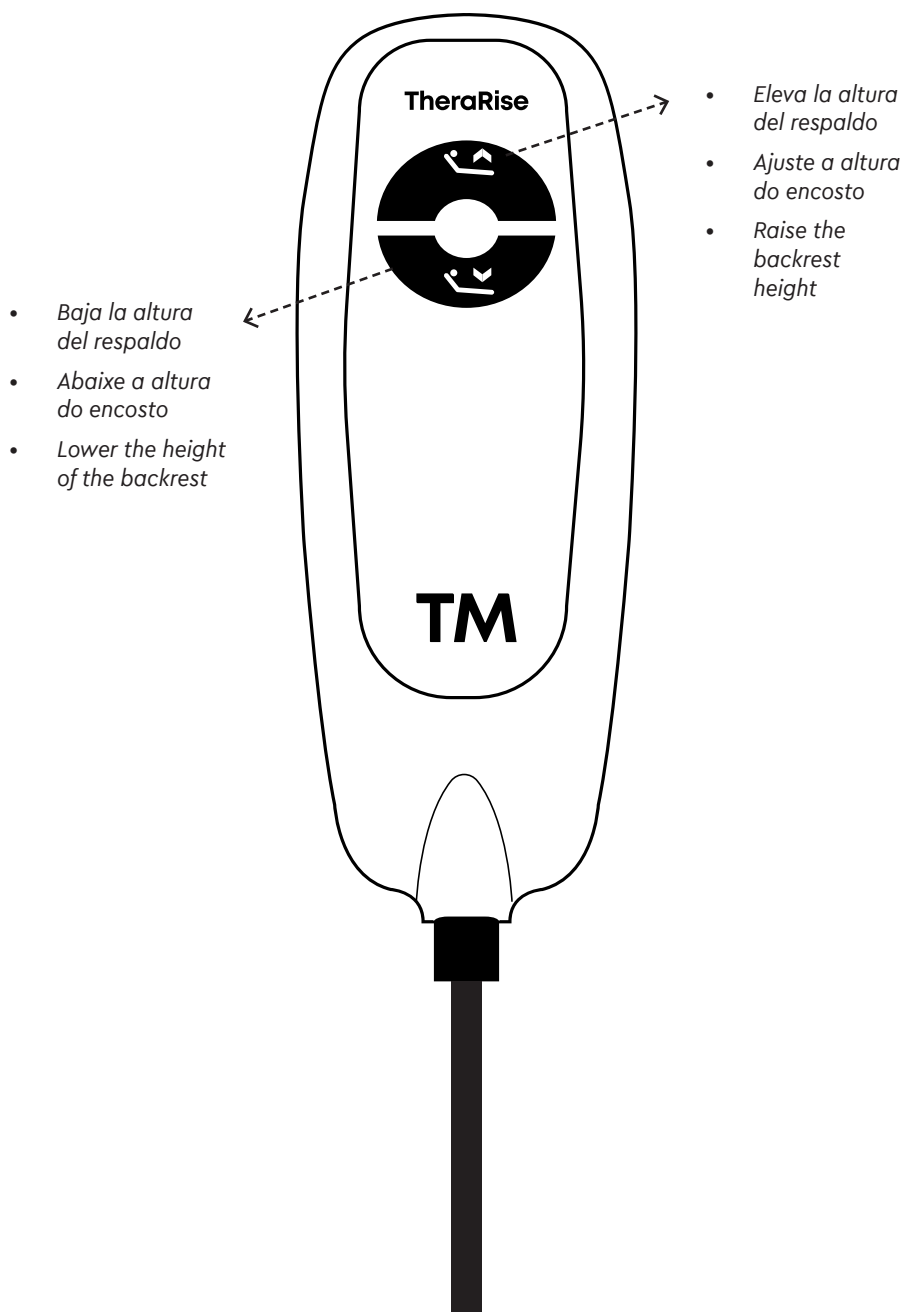
[EN]

Grupo Thera Inc. offers a 2-year limited warranty against manufacturing defects from the date of purchase. To make the warranty valid, please contact the official representative who provided the product. It is recommended to use the product in accordance with the instructions in this manual, as misuse, normal wear and tear, and damage caused by improper maintenance are not covered.

GRUPO THERA

GRUPO THERA INC. – 43rd Fl., Oceania Business Plaza, Tower 2000 – Punta Pacifica, Ciudad de Panamá – Panamá. Chile: Theramart SpA – Av. Presidente Kennedy 5600 of 507 – Vitacura, Chile. Brasil/ Brazil: Theramart Brasil Ltda – CNPJ: 45.784.439/0001-90 – Rua César Augusto Dalsóquio, nº 4255, Galpão A Sala 30 – Bairro Salseiros – Itajaí – SC – CEP 88311-500 – Brasil.

Hecho en China
Feito na China
Made in China

www.theramart.com
www.theramart.com.br
www.theramart.cl

www.grupo-thera.com