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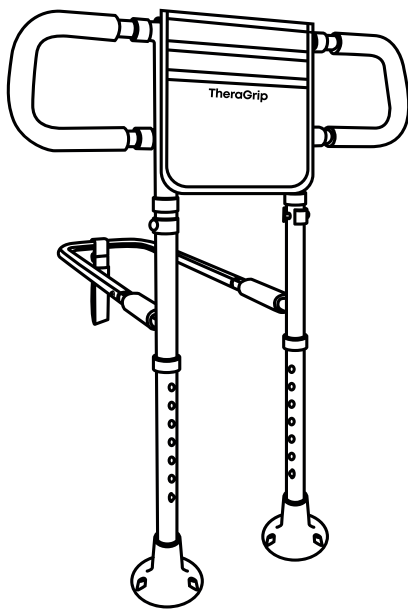
Adjustable Assistance Bar – TheraGripTM

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[EN] Thank you for choosing TheraGrip. This accessory has been designed to transform any bed into a space where autonomy is possible, where sitting up doesn't depend on someone else, where every movement is the user's choice. TheraGrip offers a secure and sturdy support point, specially designed for seniors and those spending extended periods in bed, enabling them to sit up, stabilize themselves, or shift positions with complete dignity and confidence.

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[EN] TheraGrip is an adjustable assistance bar that installs directly onto the bed, between the bed frame and mattress, with no tools or modifications needed. It provides a fixed, sturdy support point at the correct height that allows the user to sit up, position themselves, or stabilize independently. Its high-resistance steel frame, slip-resistant TPR grip, secure suction feet with TPR cups, and strap ensure safety and stability with every use. The integrated cloth bag keeps within reach what the user needs during rest.

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[EN]

- User Weight Restriction: TheraGrip is designed to support users with a maximum weight of up to 136 kg (300 lbs). Do not exceed this weight limit. Any weight exceeding this may compromise the safety and stability of the product.
- Mandatory Installation: Before using, verify that the TPR suction cups are completely adhered to the floor, flat and bubble-free. The polyester strap must be firmly secured under the mattress. These are critical safety conditions.
- Avoid placing fingers or hands in moving joints (pinch points) during height or width adjustment, especially if someone else is handling the bar.
- Keep the grip surface clean and dry. A wet or dirty grip surface may affect secure handling.
- Do not attempt to disassemble or modify any component of the bar. Height and width adjustment should only be done using the locking buttons and chrome-plated clamping system.
- Regularly inspect the suction cups, strap, and steel components for wear or deterioration. If you notice damage, discontinue use.
- Do not place objects on the bar or use it for purposes other than providing support for sitting up or stabilizing yourself.

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
[EN] Carefully remove all packaging materials. Inspect the steel frame, grip, suction cups, and strap for any visible damage that may have occurred during shipping. Ensure there is sufficient, clear space around the bed to install the bar without obstruction. Position the bar next to the bed on the side where the user needs support. Wipe the floor clean with a dry cloth if necessary to ensure the suction cups adhere properly.

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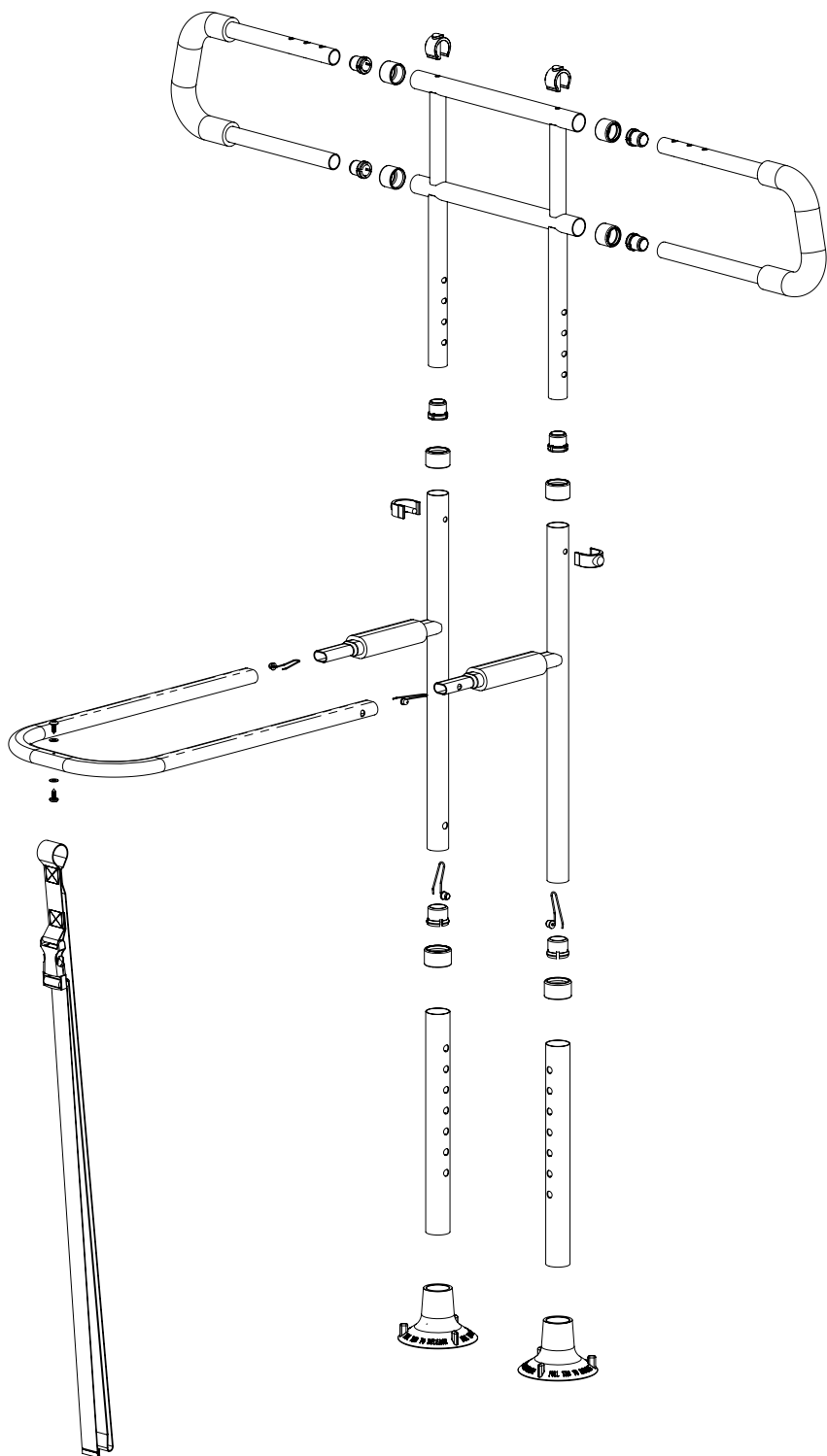[EN]

- All locking buttons are fully engaged
- Suction feet are firmly in contact with the floor
- No excessive movement when attempting to shift the bar
- Safety strap is secured under the mattress
- All plastic caps are in place

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- [EN] Test the bar gradually before first use. The user should be sitting on the bed, with the mattress in a stable position, and grip the bar gently to verify it offers the expected support. Gradually increase grip strength until you feel confident. Never load your full weight at once. Always ensure the bar doesn't move or slip before each use. If someone is assisting the user, they should be present during the first movements until both the user and assistant feel confident in its stability.

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[EN]

No Disconnection Needed / Immediate Cleaning: There is nothing to unplug. After each use, visually check that there is no dirt on the grip or suction cups.

General Grip Cleaning: For general dirt, clean the bar with a soft cloth lightly dampened with water. For more ingrained dirt, a cloth with alcohol may be used. Dry the surface immediately with a dry cloth before allowing the user to use it again.

Suction Cup Cleaning: Clean the TPR suction cups regularly with water and a soft cloth. Ensure there are no residues, dust, or lint that could prevent proper adhesion to the floor.

Strap and Bag Cleaning: The polyester strap can be cleaned with a damp cloth. The cloth bag can be hand-washed with warm water and mild soap.

Avoid Moisture: Do not allow water or moisture to come into prolonged contact with steel parts. Dry immediately after cleaning.

Prohibited Products: Do not use abrasive cleaners, solvents, or chlorine on any component of the product.

Regular Inspection: Check monthly the suction cups, strap, and all steel components for signs of wear, rust, or deterioration.

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[EN]

If the bar moves or slides: First, verify that the TPR suction cups are completely adhered to the floor and flat against the surface. Clean the floor and the underside of the suction cups with a dry cloth. If it still moves, check that the strap under the mattress is firmly secured. It may be necessary to readjust the strap length.

If the user feels unstable when gripping: Verify that the height locking buttons are fully engaged on both sides. Make sure the grip surface has no moisture or dirt that could affect grip security. Test gradually with more weight, starting from a reclined position.

If there is noise or creaking during use: This may indicate that components are not fully locked. Verify that all locking buttons are firmly engaged and that the width adjustment system is fully locked.

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[EN] At TheraMart, we create tools that support life's cycles with respect and professional quality. TheraGrip is more than just a bar: it's the point where autonomy begins, where every morning is possible without depending on others, where dignity is non-negotiable. Because when movement is secure, life is better.

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[EN]

Grupo Thera Inc. offers a 2-year limited warranty against manufacturing defects from the date of purchase. To make the warranty valid, please contact the official representative who provided the product. It is recommended to use the product in accordance with the instructions in this manual, as misuse, normal wear and tear, and damage caused by improper maintenance are not covered.

GRUPO THERA

GRUPO THERA INC. – 43rd Fl., Oceania Business Plaza, Tower 2000 – Punta Pacifica, Ciudad de Panamá – Panamá. Chile: Theramart SpA – Av. Presidente Kennedy 5600 of 507 – Vitacura, Chile. Brasil/ Brazil: Theramart Brasil Ltda – CNPJ: 45.784.439/0001-90 – Rua César Augusto Dalsóquio, nº 4255, Galpão A Sala 30 – Bairro Salseiros – Itajaí – SC – CEP 88311-500 – Brasil.

Hecho en China
Feito na China
Made in China

www.theramart.com
www.theramart.com.br
www.theramart.cl

www.grupo-thera.com